


Sterile Saline Volume Sizer Family

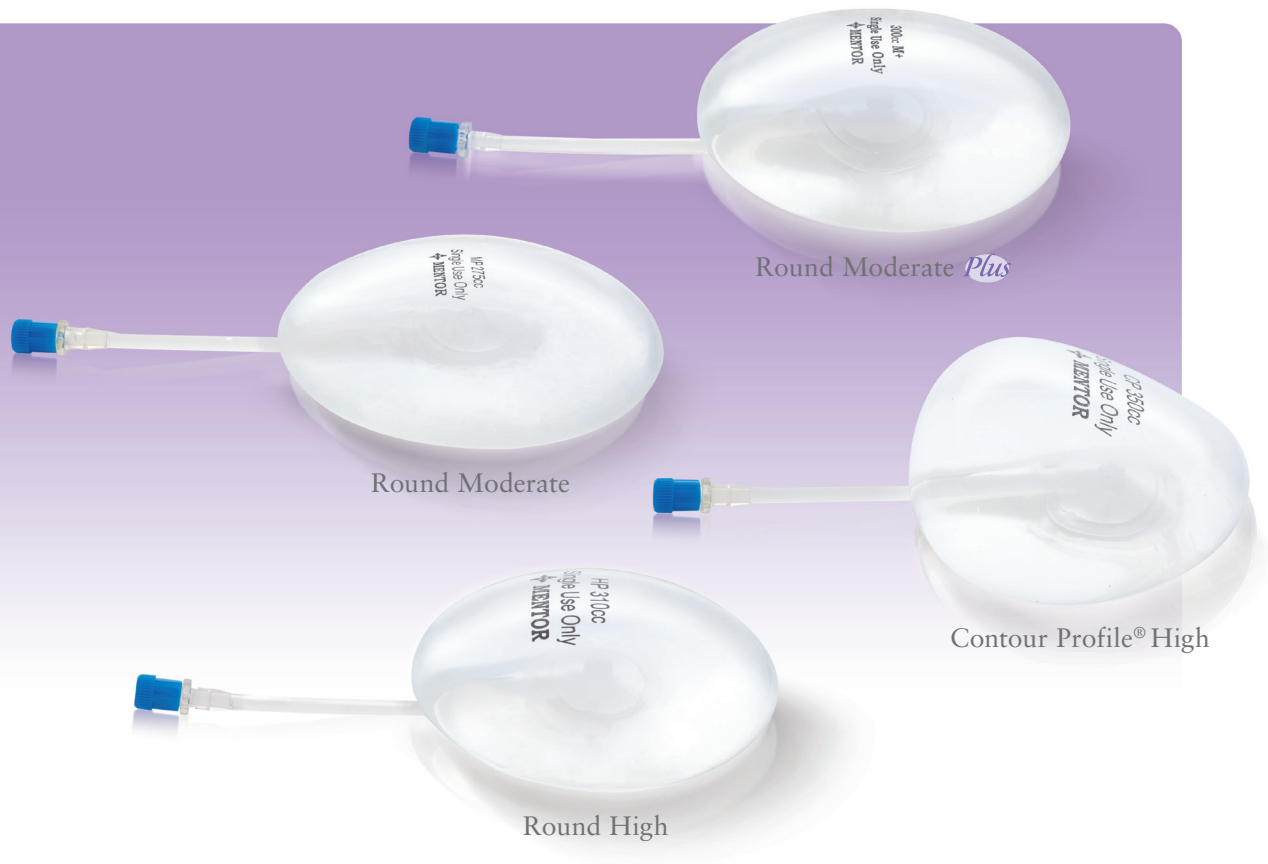

- Saline Inflatable for Volume Determination
- For Temporary Intraoperative Use
- Sterile Single Use Only
- Standard 12" Fill Tube and Reinforced Tubing To Patch Joint
- Dimensions Match our Saline Breast Implant Product Lines:
 - Round Moderate Profile
 - Round Moderate *Plus* Profile
 - Round High Profile
 - Contour Profile® High

Note: Not for permanent implantation



the power to transform®



MENTOR

Sterile Saline Volume Sizer Family

Round Moderate Profile

NOMINAL FILL			MAXIMUM FILL			
Volume	Diameter	Projection	Volume	Diameter	Projection	Catalog#
125 cc	9.5 cm	3.0 cm	150 cc	9.1 cm	3.4 cm	351-125SZ
175 cc	10.6 cm	3.3 cm	200 cc	10.0 cm	3.9 cm	351-175SZ
225 cc	11.5 cm	3.5 cm	250 cc	10.9 cm	4.0 cm	351-225SZ
275 cc	12.3 cm	3.7 cm	300 cc	11.7 cm	4.2 cm	351-275SZ
325 cc	13.0 cm	3.8 cm	375 cc	12.3 cm	4.5 cm	351-325SZ
375 cc	13.6 cm	4.0 cm	425 cc	12.8 cm	4.8 cm	351-375SZ
425 cc	14.2 cm	4.1 cm	475 cc	13.5 cm	4.9 cm	351-425SZ
475 cc	14.8 cm	4.2 cm	525 cc	14.2 cm	4.9 cm	351-475SZ

Round Moderate *Plus* Profile

NOMINAL FILL			MAXIMUM FILL			
Volume	Diameter	Projection	Volume	Diameter	Projection	Catalog#
175 cc	9.5 cm	3.5 cm	210 cc	9.3 cm	4.2 cm	351-2175SZ
200 cc	10.0 cm	3.7 cm	240 cc	9.7 cm	4.4 cm	351-2200SZ
225 cc	10.4 cm	3.8 cm	270 cc	10.1 cm	4.6 cm	351-2225SZ
250 cc	10.8 cm	4.0 cm	300 cc	10.5 cm	4.7 cm	351-2250SZ
275 cc	11.0 cm	4.1 cm	330 cc	10.8 cm	4.9 cm	351-2275SZ
300 cc	11.5 cm	4.3 cm	360 cc	11.2 cm	5.0 cm	351-2300SZ
325 cc	11.9 cm	4.4 cm	390 cc	11.5 cm	5.2 cm	351-2325SZ
350 cc	12.1 cm	4.5 cm	420 cc	11.7 cm	5.3 cm	351-2350SZ
375 cc	12.3 cm	4.6 cm	450 cc	12.0 cm	5.4 cm	351-2375SZ
400 cc	12.6 cm	4.7 cm	480 cc	12.3 cm	5.5 cm	351-2400SZ
425 cc	12.9 cm	4.8 cm	510 cc	12.5 cm	5.6 cm	351-2425SZ
450 cc	13.0 cm	4.9 cm	540 cc	12.8 cm	5.7 cm	351-2450SZ
475 cc	13.3 cm	5.0 cm	570 cc	13.0 cm	5.8 cm	351-2475SZ
500 cc	13.6 cm	5.1 cm	600 cc	13.2 cm	5.9 cm	351-2500SZ
550 cc	14.0 cm	5.3 cm	660 cc	13.7 cm	6.1 cm	351-2550SZ
600 cc	14.5 cm	5.5 cm	720 cc	14.1 cm	6.3 cm	351-2600SZ
650 cc	14.8 cm	5.6 cm	780 cc	14.4 cm	6.5 cm	351-2650SZ
700 cc	15.2 cm	5.8 cm	840 cc	14.8 cm	6.6 cm	351-2700SZ
750 cc	15.6 cm	5.9 cm	900 cc	15.1 cm	6.8 cm	351-2750SZ
800 cc	16.1 cm	6.0 cm	960 cc	15.5 cm	7.0 cm	351-2800SZ

DIRECTIONS FOR USE

1. Prior to inserting the sizer into the pocket, deflate the device completely.
2. Attach an empty, sterile syringe to the luer-lock adapter of the tube and draw out as much air as possible.
3. Fold the sizer and insert it into the pocket.
4. Infuse the sizer with sterile saline to the nominal volume.
5. After determining the proper size, add or remove saline fill with the syringe.
6. Remove the sizer and replace it with the appropriate breast implant.
7. Sizers are for single use only. Do not resterilize.





Round High Profile

NOMINAL FILL			MAXIMUM FILL			
Volume	Diameter	Projection	Volume	Diameter	Projection	Catalog #
190 cc	9.3 cm	4.1 cm	225 cc	9.1 cm	5.1 cm	351-3190SZ
230 cc	10.0 cm	4.3 cm	275 cc	9.8 cm	5.5 cm	351-3230SZ
270 cc	10.4 cm	4.6 cm	325 cc	10.2 cm	5.8 cm	351-3270SZ
290 cc	10.8 cm	4.7 cm	350 cc	10.5 cm	5.9 cm	351-3290SZ
310 cc	11.0 cm	4.8 cm	375 cc	10.7 cm	6.0 cm	351-3310SZ
330 cc	11.3 cm	4.8 cm	400 cc	11.0 cm	6.2 cm	351-3330SZ
380 cc	11.7 cm	5.2 cm	450 cc	11.4 cm	6.4 cm	351-3380SZ
420 cc	12.0 cm	5.4 cm	500 cc	11.7 cm	6.7 cm	351-3420SZ
500 cc	12.8 cm	5.6 cm	600 cc	12.4 cm	7.1 cm	351-3500SZ
630 cc	13.8 cm	5.9 cm	750 cc	13.4 cm	7.4 cm	351-3630SZ

Contour Profile® High

Volume	Width	Height	Projection	Catalog #
275+25 cc	11.5 cm	9.5 cm	5.1 cm	351-0711SZ
350+50 cc	12.3 cm	10.5 cm	5.3 cm	351-0712SZ
450+50 cc	13.2 cm	11.0 cm	6.1 cm	351-0713SZ
550+50 cc	14.0 cm	11.9 cm	6.4 cm	351-0714SZ
650+75 cc	15.0 cm	12.7 cm	6.6 cm	351-0715SZ

Note: Individual implant dimensions may vary slightly in products of this type. Not all units will conform exactly to the dimensions noted above.

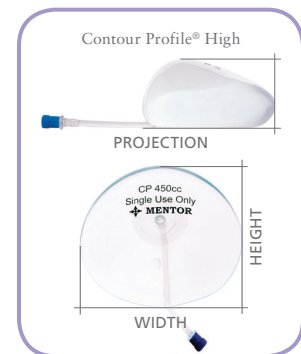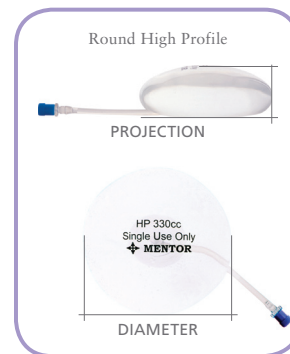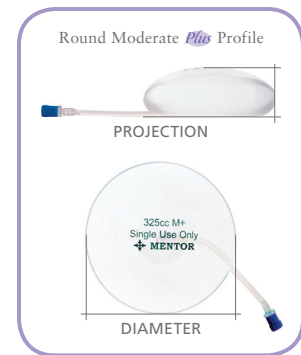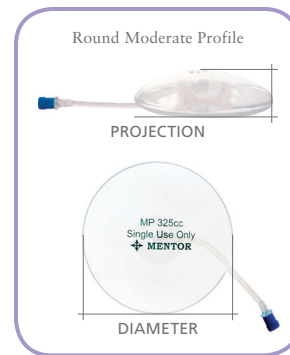

201 Mentor Drive
Santa Barbara, CA 93111 USA
(805) 879-6000

Customer Service:
(800) 235-5731
Fax: (805) 967-7108

www.mentorcorp.com

Manufacturer:
Mentor
3041 Skyway Circle North
Irving, TX 75038 USA

European Representative:
Mentor Medical Systems B.V.
2333 CL Leiden, The Netherlands

INTERNATIONAL OPERATIONS

Mentor Medical Systems, Australia, Pty. Ltd. Tel: +61 2 9251 8111 Toll Free: +1 800 338 160 Fax: +61 2 9251 8333
Mentor Medical Systems, Canada, Inc. Tel: +1 905 725 7763 Toll Free: +1 800 668 6069 Fax: +1 905 725 7340
Mentor Deutschland GmbH Tel: +49 811 600 50 0 Fax: +49 811 600 50 11
Mentor Medical Systems France, S.A. Tel: +33 1 30 07 07 60 Fax: +33 1 30 07 07 67
Mentor Medical Systems Iberica, S.L. Tel: +34 91 562 2700 Fax: +34 91 728 3367
Mentor Medical Italia, S.r.l. Tel: +39 0 2 880 7761 Fax: +39 0 2 7209 3710
Mentor Medical Systems Ltd., U.K. Tel: +44 1635 511800 Fax: +44 1635 511810

© July 2007 Mentor
0403023 Rev D